# L-Mesitran<sup>®</sup> FOAM Non-adhering foam dressing with L-Mesitran Soft

NEDERLANDS
------------

**L-Mesitran® FOAM**

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**L-Mesitran® FOAM**

**Product description**

L-Mesitran Foam is a sterile non adherent hydrophilic polyur thane foam dressing of about 3 mm, impregnated with the patented L-Mesitran Soft gel. This gel contains: 40% medical grade honey, medical grade hyppoallergenic lanolin, propylene glycol, PEG 4000 and vitamins C & E. The foam is capable of absorbing and retaining exudate.

The product creates a moist wound healing environment. This stimulates wound healing by facilitating autolysis of necrotic and other devitalised material, aiding wound bed preparation and promoting epithelial cell migration.

- L-Mesitran Foam has antibacterial properties.
- L-Mesitran Foam is a primary wound dressing that can be secured with suitable bandage.
- Due to the use of natural ingredients, each batch of product may vary in colour (white - yellow - brown).
- The gel may contain honey crystals, however this will not affect the effectiveness and/or application of the gel.

**Indications**

L-Mesitran Foam can be used in the treatment of a wide range of exuding wounds, such as (diabetic) ulcers, pressure ulcers, s perficial and partial thickness burns, fūngating wounds, donor sites, chronic wounds and post-operative wounds.
The foam dressing can be used throughout the healing process.

**Contraindications**

Do not use on patients who are known to be sensitive to the dressing or any of its components.

**Preventive measures and comments**

- Always consult a physician for proper management of the wound.
- Do not use if the pouch is damaged or already opened. Do not use after the expiry date.
- Inspect and clean the wound according to local protocol before use.
- Due to autolytic debridement the wound may appear to increase in size after the first dressing changes. This is normal and occurs as the devitalised tissue is removed, preparing the wound bed for healing.
- Each L-Mesitran Foam dressing is for single use only. Dispose of the used product as per local protocol.

**Instructions for use**

- Remove L-Mesitran Foam from the pouch and cut it to the desired size as appropriate.
- Peel off both backing sheets from the dressing and place the honey-impregnated side on the wound.

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© *Theo Manufacturing BV, February 2019*
*Theo Manufacturing B.V.*